

Privacy

Your doctor has requested that you have a PKG movement recording for several days, with particular attention paid to the times that you take your medication. Global Kinetics Corporation takes the data and develops a report including your initials, gender, doctor, age and prescribed medication and dosage. Global Kinetics Corporation takes the security of your data and personal information seriously, and protects access from unauthorised persons. Global Kinetics Corporation complies with the relevant data privacy and security regulations in your country.

Symbols and Labels for the PKG-Watch

-  Read the instructions before use
-  Serial number
-  Reference number
-  Date of manufacture
-  Manufacturer
-  Type BF applied part
-  Caution: Federal law restricts this device to sale by or on the order of a physician.
-  Keep away from heat
-  Do not dispose
-  Input power
-  IP57 Ingress protection – water and dust resistant
-  Temperature limitation - use only in temperatures between 41°F (5°C) and 104°F (40°C)

****If you do this up to 30 minutes before your prescribed medication time, the PKG-Watch registers that you have taken your medication early. The PKG-Watch will behave normally at your prescribed medication time. Slide your finger across the screen as described over to stop the vibration repeats.**

Intended Use

The Personal KinetiGraph (PKG™) System is intended to quantify kinematics of movement disorder symptoms in conditions such as Parkinson's disease, including tremor, bradykinesia and dyskinesia. It includes a medication reminder, an event marker and is intended to monitor activity associated with movement during sleep. The device is indicated for use in individuals 46 to 83 years of age.

Contraindications

None known.

Warnings

Only take your medication as prescribed by your doctor. Remove the PKG-Watch immediately and contact your clinic if you experience a reaction of the skin (e.g. irritation or itchiness) that is in contact with the PKG-Watch. Remove the PKG-Watch and contact your clinic if the PKG-Watch becomes uncomfortably warm. Do not use the PKG-Watch in temperatures greater than 104°F (40°C) and less than 41°F (5°C). Do not leave the PKG-Watch in direct sunlight or in any place where it might become hot. Do not use if there are signs of physical damage or modification. The PKG-Watch is suitable for use in all home and domestic environments. Interference to normal operation of the PKG watch may occur if it is operated next to other electrical equipment, hospital electrical equipment or industrial electrical equipment. Do not use closer than 30cm (12 inches) of radio communications equipment. The PKG-Watch must not be able to move on the wrist or be secured too tightly. Do not modify any parts of the PKG-Watch.

Precautions

The PKG™ is to be used and interpreted only by trained clinicians and technicians. Only use the PKG-Watch as instructed by your clinician.

Cleaning

If the PKG-Watch is soiled, clean with warm soapy water

KinetiGraph™ and PKG™ are registered trademarks of Global Kinetics Corporation..

Rx Only



PKG™-Watch Patient Guide

For Personal KinetiGraph™

Clinic Contact Name: _____

Clinic Phone: _____

Wrist: left / right (circle appropriate)

Distributed by

Global Kinetics USA Corporation
5810 West 78th Street,
Suite 210
Bloomington, MN, 55439, USA
www.globalkineticscorporation.com




GKC Manufacturing Pty Ltd
Level 9, 31 Queen Street
Melbourne, VIC, 3000, Australia

Your doctor has referred you for a Personal KinetiGraph™ (PKG™). The PKG-Watch records your movement for 6-10 days. When the recording is finished, return the watch for the results to be sent to your doctor. The results show your doctor how your movement changes throughout the day.


This watch is ready for you to wear. Put on the watch now.

- Wear it directly against your skin.
- Wear it on the side most affected by your Parkinson's.
- Wear it on the same wrist all day and night for 6-10 days.

The watch is waterproof for up to 30 minutes.

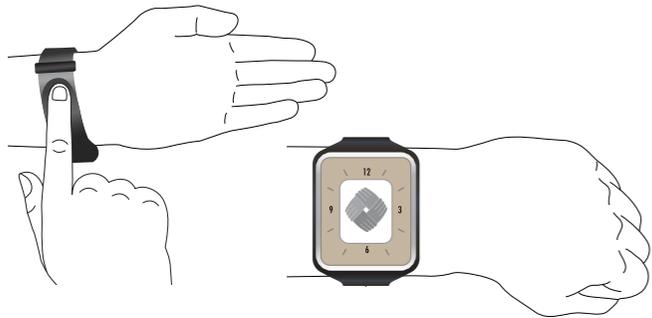

If the watch displays a logo, hold 2 fingers on the screen for at least 10 seconds until the clock appears. The watch is recording when it displays the clock.

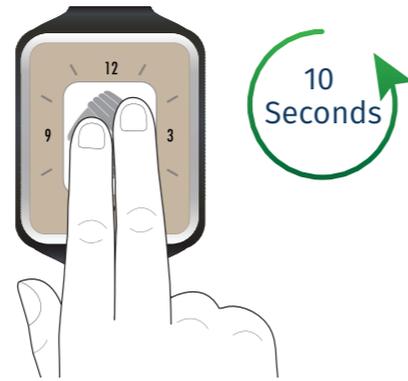


The watch may immediately vibrate 5 times. This is a demonstration of the medication reminders to allow you to practice. After the practice, the watch will only vibrate at your prescribed Parkinson's disease medication times*.

*Your watch may not be programmed with a demonstration or medication reminders. Please continue to wear the watch and take your Parkinson's disease medications at the times prescribed by your doctor.

At your prescribed Parkinson's disease medication times, your watch will vibrate. The pill and the orange light will appear on the screen.




Immediately after you've taken your Parkinson's disease medication, **slowly** slide your finger across the screen from 9 to 3 until the tick is displayed and the green light appears**.

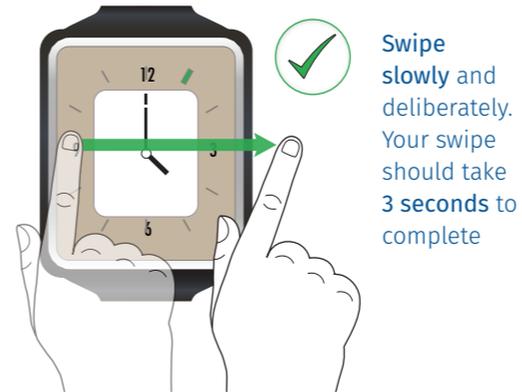

Swipe **slowly** and deliberately. Your swipe should take **3 seconds** to complete

If you don't successfully swipe, the vibration will repeat and the pill and the orange light stay on the screen.

When the recording is finished an envelope will appear on the screen.

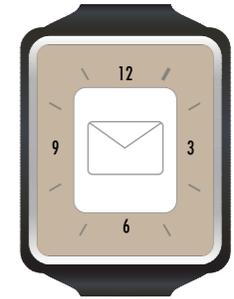

Return the watch to your PKG clinic. The results will be taken from the watch and sent to your doctor.

TIP: You can light the screen of the watch. Hold 2 fingers on the screen briefly, then remove. The backlight will automatically turn off.

⚠ Only take your medication as prescribed by your doctor.

If you have any questions about how to operate the watch or if it behaves unexpectedly, contact your PKG clinic.